

# Community Face Mask

The **Community Face Mask** is a single-use face mask designed for public or community spaces where social distancing cannot be achieved.

Featuring three layers of lightweight material with two ear loops to keep the mask in place, this mask provides comfort and good breathability for the wearer.

The Community Face Mask can be used in retail, food service and transport environments, as well as for events, festivals and conferences.

This mask is not intended for use in clinical environments.

- + Australian Made and Owned
- + Single-use face masks with ear loops
- + Inner comfort layer for comfortable fit
- + Non-sterile and Latex-free



## Testing

We operate to Quality Management System ISO: 13845. Raw Materials and Finished Goods are tested routinely in Detmold Medical's validated state-of-the-art laboratory.


## Applications

- + Crowded spaces (events, retail, hospitality)
- + Public transport and travel
- + Everyday use in the community

## Material Specifications

### Outer Layer

The external hydrophobic layer provides protection for the filtration layer.

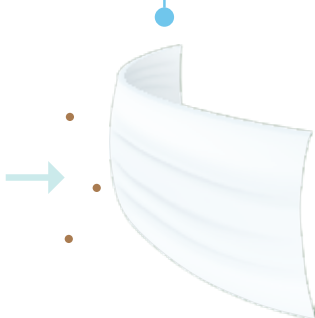


### Middle Layer

The meltblown filtration layer protects from the passage of non-oil based particulates outside the mask.



### Inner Layer

Soft and comfortable, the inner hydrophilic (absorbent) layer captures droplets from wearer.

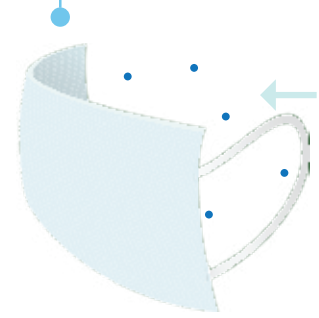

● Droplets from wearer ● Non-oil based particulates



# Community Face Mask

## Product Details

Item No.	Description	Size	Qty Per Box	Qty Per Carton	Shipper Weight	Shipper dimensions
W401M	3-Ply Community Face Mask	One size	50	1200	6kg	415 x 635 x 259mm
W218M	3-Ply Community Face Mask 10-Pack. Made to order only. MOQ 300		30 x 10	300	3kg	216 x 447 x 209mm

## Instructions for Use

**Applying the Mask** | Prior to donning or doffing, ensure hand hygiene is undertaken. 

Once the mask is applied it should not be touched or re-used. Please discard safely after use.



1. Hold the mask on the short edges with the ear loop connection points facing outward.



2. Hold firmly on the nose piece, place one ear loop around the back of your ear and ensure it is secure.



3. Place the second ear loop up over your other ear and check the mask is evenly aligned on your face.

4. Expand the pleats to ensure a secure and comfortable fit.



5. Check mask fit to ensure it is secured at the bridge of the nose and mouth. There should be minimal gaps between the mask and face.

## Storage Recommendations

Store mask in the original packaging, away from contaminated areas, dust, sunlight, extreme temperatures, excessive moisture, and damaging chemicals. Storage temperature -20°C to +30°C at <80% relative humidity. When stored in accordance with recommendations, the product is designed for use within 2 years from date of manufacture.



**Caution:** Misuse may result in sickness or death. DO NOT use in any manner not indicated in the Instructions for Use. Always read the label. Discard the mask if it becomes damp, soiled, damaged, or difficult to breathe through. Do not use if the packaging is damaged.

Contact Detmold Medical to place an order today

©2024 Detmold Medical. All Rights Reserved. February 2024. #DM-MCOM-2024-002



✉ [connect@detmoldmedical.com](mailto:connect@detmoldmedical.com)

📞 1800 338 633

[detmoldmedical.com](http://detmoldmedical.com)